3M ESPE Protemp™ II

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3M ESPE, Protemp and RelyX are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2020, 3M. All rights reserved.

<div>en</div>	ENGLISH
<div>de</div>	DEUTSCH
<div>fr</div>	FRANÇAIS
<div>it</div>	ITALIANO
<div>es</div>	ESPAÑOL
<div>pt</div>	PORTUGUÊS
<div>nl</div>	NEDERLANDS

Protemp II is a material for the fabrication of temporary restorations inside the patient's mouth. The 3-component system based on multi-functional methacrylate esters has filling material-like properties. The available colours are A1 extra-bright, A3 bright, and B3 yellow.

☛ Instructions for Use should not be discarded for the duration of product use.

- Indications**
 - Fabrication of temporary crowns, bridges, inlays, and onlays.
- Contraindications**
 - The product is contraindicated for use with patients who have a known allergy to acrylic based dental material or contained allergens as listed in section "Precautionary Measures".

Potential Undesirable Side Effects and Complications
Potential effects are local irritations, or local or systemic allergic reactions. Retention loss or fracture of the restoration can lead to swallowing or aspiration of the restoration or parts of it in rare cases.

Precautionary Measures
For Patients

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylic allergies.
- If prolonged contact with oral tissue occurs, flush with large amounts of water.
- In allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use.

For Dental Personnel

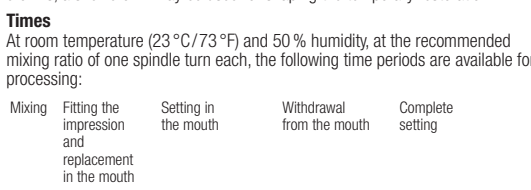
- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.
- The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the product contacts the glove, remove and discard the glove, wash the hands immediately with soap and water and then re-glove.
- If allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Take an alginate or silicone impression of the intact row of teeth.
- To enhance the strength of the temporary restoration, remove inter-dental grooves. If required due to tight spacing conditions, enlarge the relevant sites by cutting.
- For easy replacement in the mouth relieve undercuts.
- With missing teeth or gaps in the molar area, cut a groove for a stable bar-shaped connector in the temporary restoration.
- If required, prior to taking the impression close gaps between front teeth with ready-made teeth acting as spacers. Interlock multiple teeth with wax. Instead of an impression, a laboratory-made deep drawing template or, for single crowns, a shell crown may be used for shaping the temporary restoration.

Times
At room temperature (23 °C/73 °F) and 50% humidity, at the recommended mixing ratio of one spindle turn each, the following time periods are available for processing:

Mixing and replacement in the mouth	Fitting the impression in the mouth	Setting in the mouth	Withdrawal from the mouth	Complete setting
				

Dosing and Mixing

- For the fabrication of a temporary crown or bridge unit dispense one full turn each of the dosing syringe spindle of base and catalyst paste onto the mixing block. The locking in position of the spindle after one full turn produces a clearly audible "click" sound.
- To accelerate the setting reaction, the amount of catalyst may be increased up to two-to1 (**max.1**). With increased quantities of catalyst, the processing and withdrawing times are shortened! Further over-dosing or under-dosing changes the mechanical properties of the material (and increases the risk of fracturing).
- Re-cap the dosing syringes carefully after use.
- Take up the catalyst paste with a spatula and mix with base paste by repeatedly streaking out the pastes on the block until the mixture is homogeneous.

Application

- Fill the mixture of pastes into the Protemp™ syringe and load the dried impression or deep drawing template in the relevant spaces from bottom up.
- Replace the impression in the mouth.
- At the recommended dosages (1:1 turns of the spindle) the material attains a tough-elastic consistency approx. 4 min after the onset of mixing. The material and the impression or deep drawing template must be removed from the mouth within approx. 6 min after the onset of mixing. Control the setting process by means of the excess in the mouth.**

Finishing

- Remove the superficial inhibition layer formed by the oxygen in air with alcohol (e.g., ethanol) once the material is completely set (not earlier than 7 minutes after the onset of mixing).
- If required, shape the temporary restoration. Do not inhale polishing dust; wear a protective mask or goggles or use an aspirator.

Cementation

- Insert temporary restorations with temporary temporary fixation cements, e.g. RelyX™ Temp E or RelyX™ Temp NE. Eugenol-containing cements may impair the setting of fixation composites to be used at a later point in time.

After Cementation

- Thoroughly examine the sulcus of the prepared teeth and surrounding dentition. Remove any residual temporary material or cement from the mouth.

Repairs

- Fragments of restorations broken-off soon after fabrication may be re-bonded with freshly mixed Protemp II.
- With **broken temporary restorations that have been in situ** for a while, provide for mechanical retention by roughening the surfaces of the breaks and adjacent areas. If required, make undercuts. Light-curing bonding materials can strengthen the bond. Repair the temporary restoration outside the mouth with freshly mixed Protemp II or a methylethacrylate or, alternatively, use a composite material for repairs inside the mouth.

Cleaning the Instruments

- Paste that has not set can be removed with organic solvents.

Multiple Use of Medical Device
Before use of the dosing syringes, put on new, unused disposable gloves during all clinical applications of the dosing syringes to eliminate the risk of contamination!

Protemp II dosing syringe is not visibly clean after use, i.e., it, contrary to expectations, there is contamination with blood or saliva during the treatment, for example, discard the dosing syringe immediately.

Inspection, Maintenance, Testing

- Examine the medical device for damage, discoloration, and contamination before every use.
- Do not use damaged and contaminated devices under any circumstances.

Notes
The application syringe is a single-use product. There is no procedure for clean-ing, disinfecting, and sterilizing the application syringe that would guarantee the safe use of multiple times of the syringe.

Disposal
Dispose of the contents or container in accordance with the applicable regula-tions. Please pay special attention to the disposal of contaminated waste to avoid health risks because of improper handling.

Storage and Stability
Store the product at 15–25 °C/59–77 °F. Do not use after the expiry date.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability
Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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