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3M, ESPSE, Filtek, Garant, Pantem, Protemp, ReMix и Sof-Lex are trademarks of 3M or 3M Deutschland GmbH. © 2022, 3M. Unauthorized use prohibited. All rights reserved.

ENGLISH

**Product Description**  
This product is intended for the fabrication of temporary restorations using a multi-functional methacrylate esters, this two-component system has filling material properties and is available in the following colors: A1, A2, A3, A3.5, B3, Bleach. With the Garant Dispenser, the material can be completely removed without causing any bubbles to form. Protemp 4 temporary restorations can be modified and individualized using low-viscosity sticky resin (3M or 3M Deutschland GmbH product line).  
\*Not available in all countries.

These Instructions for Use should be kept for the duration of product use. The product may only be used when the product labeling is clearly readable. For details on all additionally mentioned products please refer to the Instructions for Use.

**Intended Purpose**  
Intended purpose: temporary restorative material for temporary restoration on prepared teeth.  
Intended users: educated dental professionals who have theoretical and practical knowledge of dental restorative materials and procedures.  
Clinical benefit: temporary restoration of oral esthetics and function.  
Patient target group: all patients requiring dental treatment, unless the product information limits the use.

**Indications**  
► Fabrication of temporary crowns, bridges, inlays, onlays, and veneers  
► Fabrication of long-lasting temporary restorations  
► Lining material for prefabricated temporary crowns made of composite resin (3M or 3M Deutschland GmbH product line).

**Contraindications**  
The product is contraindicated for use with patients who have a known allergy to acrylate based materials or contained allergens as listed in section "Precautionary Measures".

**Composition**  
This product consists of each component of the product are indicated in decreasing order of their concentration.  
Base Paste: Dimethacrylate (Bis-MEPP), silane treated silica, methacrylated polyethylene glycol (PMDEGA), photoinitiator.  
Catalyst Paste: Diacetate, barbituric acid derivative, silane treated silica, polyester.

For more information, please contact the Safety Data Sheet/Safety Information Sheet (www.3m.com) or contact your 3M subsidiary.

**Precautionary Measures**  
► **For Patients**  
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product if skin contact occurs, wash skin with soap and water. The use of protective gloves and other protective techniques is recommended. Acrylics are not commonly used gloves. If the product contains the glue, remove and discard the glue, wash the hands immediately with soap and water, and then re-prepare. If allergic reaction occurs, seek medical attention.  
Please report a serious incident occurring in relation to 3M and the local competent authority (EU) or local regulatory authority.

**Potential Undesirable Side Effects and Complications**  
Potential effects are local irritations, or local or systemic allergic reactions. Reaction loss or fracture of the restoration can lead to swelling or aspiration. Do not use after expiration date.  
The summary of safety and clinical performance for the device can be reviewed in the European database for medical devices (Eudamed) under the following link (as soon as the corresponding database module is available): <https://ec.europa.eu/tools/eudamed>  
The Instructions for Use and the summary of safety and clinical performance are also available at: <https://hcbg.orgulatory.3m.com>  
Basic UID-DE: 0608223276102000000400N

**Preparatory Work**  
► Take an alginate, silicone, or polyether impression from the intact crown of teeth.  
► Instead of an impression, a laboratory-made vacuum-formed template or, for single crowns, a preformed crown can be used for shaping.  
► To enhance the strength of the impression, use a vacuum-forming template grooves in the impression. If required due to lifting spacing conditions, enlarge the relevant sites by cutting.  
► For easy replacement in the mouth, include undercuts.  
► With missing teeth or gaps in the molar area, cut a groove into the impression to produce a stable, bar-shaped connection in the temporary restoration.  
► Prior to taking the impression, close any gaps between front teeth with denture teeth acting as spacers; interlock multiple denture teeth with wax.  
► Do not use hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) to prepare the tooth, as the residue can adversely affect the setting reaction.  
► Select a Protemp cartridge of the desired color and place it ready for use.

**Times**  
00:00–00:40 min:sec Place Protemp 4 into the impression or into the matrix/model and reposition in the mouth  
01:40–02:50 min:sec Setting in the mouth or in the matrix/model  
01:40–02:50 min:sec The time for the removal of the impression or of the matrix/model

Up to 05:00 min:sec Complete setting  
From 05:00 min:sec Removal of the temporary restoration from the impression or from the matrix/model, finishing

The above processing times are applicable to products which have been stored and processed at a temperature of 23 °C/74 °F at 50% relative humidity. Higher temperatures shorten the times, while lower temperatures extend them. The size of the restoration also has an impact on the setting times; as the size of the workpiece increases, the setting accelerates.

**Dosing and Mixing**  
► Dispense and statically mix in the Garant dispenser.  
Use the enclosed original Garant Mixing Tips Blue only.  
**During the initial dosing**  
► **Remove and discard the cartridge lid.** Under no circumstances should the cartridge lid be used to release the cartridge as this could result in the inclusion of air bubbles.  
► Check whether both cartridge openings are free from obstruction and remove obstructing material, if any, with a suitable instrument.  
► Before a mixing tip is applied for the first time, squeeze paste out onto the mixing tip until the base and catalyst paste are at the same level.  
Discard the paste which has been squeezed out.

**Application**  
It takes the material a certain amount of time to emerge from the newly activated mixing tip. The flow of material is interrupted as soon as the pressure

on the handle is reduced. Material that has set into the mixing tip should not be extruded by force, as this may damage both the cartridge and the mixing tip. Core build-up which has formed on the mixing tip should be removed using a modified glassionomer cements must be sealed off before the application of Protemp 4, e.g., using Vaseline.

► First, install a mixing tip.  
► Prior to each application, extrude a small (pea-sized) amount of paste from the mixing tip onto a mixing pad and discard this material.  
► Subsequently load the dred impression or vacuum-formed template in the relevant spaces from the bottom up.  
► Reposition the impression or vacuum-formed template in the mouth/on the matrix/model.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing. The material and the impression or vacuum-formed template must be removed from the mouth/model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the excess material in the mouth or on the matrix/model.  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.

► **Finishing**  
► Completely remove the oxygen inhibition layer using alcohol (e.g., ethanol) once the material is completely cured (not earlier than 5 minutes after the onset of mixing). Polish if desired, e.g., with Sof-Lex polishing discs.  
After the product has cured completely, remove any residue.  
► If necessary, finish and polish the temporary restoration using fine carbide burs.  
► Use a fine abrasive polishing dust. Use mouth protective device, safety glasses and breathe.

**Individualization/Shape Correction**  
Using a Filtek Flowable Composite and Protemp 4 for custom-shaping and shape correction, there usually is no need for a bonding agent. If you intend to use other products, please refer to the pertinent manufacturer's instructions for use.

**Compatible Products**  
► Flowable composites, e.g., from the Filtek® Flowable Composite product line: A1, A2, A3, A3.5, B2, WX (Bleach)  
► Products compatible with the corresponding Instructions for Use!  
► Protemp 4  
► Processing: please refer to the information provided under "Application" and "Finishing".

**Application of a Filtek Flowable Composite to New Temporary Restorations**  
In newly manufactured temporary restorations, the Filtek Flowable Composite can be applied either to the finished surface or the unprocessed, but clean, preparation surface depending on the application.  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol, then dry with a stream of air. Cleaning is not necessary if the restoration is already finished and ready for use.  
For reasons of hygiene, dose the Filtek Flowable Composite from the dispenser onto a mixing pad.  
► Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument.  
► Light-cure each layer separately for 20 sec.  
► Trim the composite with carbide burs and polish, e.g., with Sof-Lex polishing discs.

**Application of a Filtek Flowable Composite to Temporary Restorations That Were in Use**  
► Thoroughly roughen the surface of the temporary restoration with rotary instruments.  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol, then dry the restoration with a stream of air.  
For reasons of hygiene, dose the Filtek Flowable Composite from the dispenser onto a mixing pad.  
► Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument.  
► Light-cure each layer separately for 20 sec.  
► Trim the composite with carbide burs and polish, e.g., with Sof-Lex polishing discs.

**Lining of Prefabricated, Temporary Crowns Made of Composite and Metal Core build-ups**  
These are materials which are fabricated using composites, composites, or resin-veneered composites. The material must be sealed off before the application of Protemp 4, e.g., using Vaseline.  
► Shape and cure prefabricated, temporary composite crowns (e.g., Protemp Crown) completely.  
► Please comply with the corresponding Instructions for Use!  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol (e.g., ethanol), then dry with a stream of air.  
► Apply Protemp 4 to the crown in a multiple patient multiple use device.  
► Place the prefabricated crown on the prepared tooth/ide.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing and must be removed, along with the prefabricated crown, from the mouth/model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the residue in the mouth or in the matrix/model.  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.  
► Remove the oxygen inhibition layer and finish the residues, see "Finishing".

**Cementation**  
► If the prepared tooth or restoration has been sealed off (e.g., using Vaseline), remove the rest of the sealant thoroughly before the temporary cementation.  
► Insert the temporary restoration with a commercial temporary cement, e.g., ReMix Temp E or ReMix Temp NE. If the application is used as a temporary temporary restoration, we recommend using a stronger cement such as ReMix Temp C.  
► Eugenol-containing cements may impair the setting of composite luting agents. Do not use after the expiration date.  
► The material is to be used for the final luting step, see the temporary restoration with a eugenol-free cement.

**After Cementation**  
► Finish the crown and explore the spaces and surrounding dentition.  
► Remove any residuals of the temporary material or cement from the mouth.  
**Fabrication of a Temporary Restoration in the Laboratory**  
**Model Preparation**  
► Take an impression to prepare an unsaved work model. If necessary, use a wax-up (wax modeling). Prepare a silicone matrix using a laboratory model.  
► Remove the matrix and wax-up from the model, e.g., using a steam cleaner or boiling water.  
► The use of a suitable dental wax to block out the undercuts on the work model will make the removal of the temporary restoration from the model easier. Apply a thin layer of wax to the dies in addition. As necessary, cut out interlocking grooves and constrictions in the gingiva area in the matrix.  
► Treat the model with a suitable plaster composite isolation in accordance with the applicable instructions for use.

**Preparation of the Temporary Restoration**  
► Install a new mixing tip.  
► Prior to each application, extrude a small (pea-sized) amount of paste from the mixing tip onto a mixing pad and discard this material.  
► Subsequently load the dred impression/matrix in the relevant spaces from the bottom up.  
► Reposition the impression/matrix or the vacuum-formed template on the model.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing. The material and the impression/matrix or vacuum-formed template must be removed from the model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the excess material on the matrix/model.  
► Do not wait until the temporary restoration has hardened completely before removing it (see "Times").  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.

**Finishing the Temporary Restoration**  
► To obtain an optimal composite surface, leave the temporary restoration in the matrix for at least 5 min after start of mixing, then remove the matrix.  
► Higher temperatures shorten the times, while lower temperatures extend them. The size of the restoration also has an impact on the setting times; as the size of the workpiece increases, the setting accelerates.  
► Do not use additional curing methods; they would cause excessive shrinkage. **Do not use heat, pressure, or light for any additional curing of the temporary restoration.**  
Information about finishing and polishing can also be found in "Finishing" and "Individualization/Shape Correction". **Do not clean the temporary restoration with steam** because this can lead to problems with the fitting. Do not use alcohol (e.g., ethanol) or disinfectants containing alcohol to clean the temporary restoration.

**Note: Colored occlusal spray can cause discoloration of the temporary restoration!**  
**Storage and Transport**  
► Store the material in the original packaging.  
► Store the temporary restoration by placing on 3M Deutschland GmbH and transporting to the model.

**Repair**  
► Provide for mechanical and chemical retention by roughening the break surfaces and adjacent areas.

► Clean with alcohol and dry with air.  
► Join together using Protemp 4 or a Filtek Flowable Composite.  
► Cure the material using a laboratory-made vacuum-forming template or a modified glassionomer cements must be sealed off before the application of Protemp 4, e.g., using Vaseline.

► First, install a mixing tip.  
► Prior to each application, extrude a small (pea-sized) amount of paste from the mixing tip onto a mixing pad and discard this material.  
► Subsequently load the dred impression or vacuum-formed template in the relevant spaces from the bottom up.  
► Reposition the impression or vacuum-formed template in the mouth/on the matrix/model.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing. The material and the impression or vacuum-formed template must be removed from the mouth/model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the excess material in the mouth or on the matrix/model.  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.

► **Finishing**  
► Completely remove the oxygen inhibition layer using alcohol (e.g., ethanol) once the material is completely cured (not earlier than 5 minutes after the onset of mixing). Polish if desired, e.g., with Sof-Lex polishing discs.  
After the product has cured completely, remove any residue.  
► If necessary, finish and polish the temporary restoration using fine carbide burs.  
► Use a fine abrasive polishing dust. Use mouth protective device, safety glasses and breathe.

**Individualization/Shape Correction**  
Using a Filtek Flowable Composite and Protemp 4 for custom-shaping and shape correction, there usually is no need for a bonding agent. If you intend to use other products, please refer to the pertinent manufacturer's instructions for use.

**Compatible Products**  
► Flowable composites, e.g., from the Filtek® Flowable Composite product line: A1, A2, A3, A3.5, B2, WX (Bleach)  
► Products compatible with the corresponding Instructions for Use!  
► Protemp 4  
► Processing: please refer to the information provided under "Application" and "Finishing".

**Application of a Filtek Flowable Composite to New Temporary Restorations**  
In newly manufactured temporary restorations, the Filtek Flowable Composite can be applied either to the finished surface or the unprocessed, but clean, preparation surface depending on the application.  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol, then dry with a stream of air. Cleaning is not necessary if the restoration is already finished and ready for use.  
For reasons of hygiene, dose the Filtek Flowable Composite from the dispenser onto a mixing pad.  
► Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument.  
► Light-cure each layer separately for 20 sec.  
► Trim the composite with carbide burs and polish, e.g., with Sof-Lex polishing discs.

**Application of a Filtek Flowable Composite to Temporary Restorations That Were in Use**  
► Thoroughly roughen the surface of the temporary restoration with rotary instruments.  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol, then dry the restoration with a stream of air.  
For reasons of hygiene, dose the Filtek Flowable Composite from the dispenser onto a mixing pad.  
► Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument.  
► Light-cure each layer separately for 20 sec.  
► Trim the composite with carbide burs and polish, e.g., with Sof-Lex polishing discs.

**Lining of Prefabricated, Temporary Crowns Made of Composite and Metal Core build-ups**  
These are materials which are fabricated using composites, composites, or resin-veneered composites. The material must be sealed off before the application of Protemp 4, e.g., using Vaseline.  
► Shape and cure prefabricated, temporary composite crowns (e.g., Protemp Crown) completely.  
► Please comply with the corresponding Instructions for Use!  
► Remove any contamination – e.g., from saliva or polishing dust – with alcohol (e.g., ethanol), then dry with a stream of air.  
► Apply Protemp 4 to the crown in a multiple patient multiple use device.  
► Place the prefabricated crown on the prepared tooth/ide.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing and must be removed, along with the prefabricated crown, from the mouth/model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the residue in the mouth or in the matrix/model.  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.  
► Remove the oxygen inhibition layer and finish the residues, see "Finishing".

**Cementation**  
► If the prepared tooth or restoration has been sealed off (e.g., using Vaseline), remove the rest of the sealant thoroughly before the temporary cementation.  
► Insert the temporary restoration with a commercial temporary cement, e.g., ReMix Temp E or ReMix Temp NE. If the application is used as a temporary temporary restoration, we recommend using a stronger cement such as ReMix Temp C.  
► Eugenol-containing cements may impair the setting of composite luting agents. Do not use after the expiration date.  
► The material is to be used for the final luting step, see the temporary restoration with a eugenol-free cement.

**After Cementation**  
► Finish the crown and explore the spaces and surrounding dentition.  
► Remove any residuals of the temporary material or cement from the mouth.  
**Fabrication of a Temporary Restoration in the Laboratory**  
**Model Preparation**  
► Take an impression to prepare an unsaved work model. If necessary, use a wax-up (wax modeling). Prepare a silicone matrix using a laboratory model.  
► Remove the matrix and wax-up from the model, e.g., using a steam cleaner or boiling water.  
► The use of a suitable dental wax to block out the undercuts on the work model will make the removal of the temporary restoration from the model easier. Apply a thin layer of wax to the dies in addition. As necessary, cut out interlocking grooves and constrictions in the gingiva area in the matrix.  
► Treat the model with a suitable plaster composite isolation in accordance with the applicable instructions for use.

**Preparation of the Temporary Restoration**  
► Install a new mixing tip.  
► Prior to each application, extrude a small (pea-sized) amount of paste from the mixing tip onto a mixing pad and discard this material.  
► Subsequently load the dred impression/matrix in the relevant spaces from the bottom up.  
► Reposition the impression/matrix or the vacuum-formed template on the model.  
► The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing. The material and the impression/matrix or vacuum-formed template must be removed from the model within 2 min 50 sec after the onset of mixing.  
► Check the setting process by observing the excess material on the matrix/model.  
► Do not wait until the temporary restoration has hardened completely before removing it (see "Times").  
► Keep the filled mixing tip on the cartridge to serve as a closure until the next use.

**Finishing the Temporary Restoration**  
► To obtain an optimal composite surface, leave the temporary restoration in the matrix for at least 5 min after start of mixing, then remove the matrix.  
► Higher temperatures shorten the times, while lower temperatures extend them. The size of the restoration also has an impact on the setting times; as the size of the workpiece increases, the setting accelerates.  
► Do not use additional curing methods; they would cause excessive shrinkage. **Do not use heat, pressure, or light for any additional curing of the temporary restoration.**  
Information about finishing and polishing can also be found in "Finishing" and "Individualization/Shape Correction". **Do not clean the temporary restoration with steam** because this can lead to problems with the fitting. Do not use alcohol (e.g., ethanol) or disinfectants containing alcohol to clean the temporary restoration.

**Note: Colored occlusal spray can cause discoloration of the temporary restoration!**  
**Storage and Transport**  
► Store the material in the original packaging.  
► Store the temporary restoration by placing on 3M Deutschland GmbH and transporting to the model.

**Repair**  
► Provide for mechanical and chemical retention by roughening the break surfaces and adjacent areas.

Reference Number and Symbol	Symbol	Description of Symbol
CE Mark		Indicates conformity to all applicable European Union Medical Device Regulations and Directives.
ISO 15223-1 5.7.7 Medical Device Class		Indicates the item is a medical device.
Corrugated fibero-band		Indicates product packaging is made of corrugated fibero-band. Official Journal of the EC, Commission Decision (97/129/EC)
Non-corrugated fibero-band		Indicates product packaging is made of non-corrugated fibero-band. Official Journal of the EC, Commission Decision (97/129/EC)
Low-density polyethylene		Indicates product packaging is made of low-density polyethylene. Official Journal of the EC, Commission Decision (97/129/EC)
Green Dot		Indicates a financial contribution to national packaging recovery company per European Directive No. 94/62 and corresponding national law. Packaging Regulatory Organization Europe.
ISO 15223-1 5.1.8 Importer		Indicates the entity importing the medical device into the locale.
For more information, see: HCBGRegulatory.3m.com		Information valid as of: July 2022

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